

# Wymo Wheelchair Cover Instructions



## STEPS:

### To Load the Wheelchair with Cover

- Transfer from the wheelchair into the driver's seat, fold the wheelchair, rotate the wheelchair onto its handles
- Place the weather cover over the chair, with the rectangular window facing towards you
- Ensure that the cover is over the bottom section of the wheelchair (you can pull the cover down as you raise the wheelchair)
- **Press** down on the switch box until the hook can be correctly attached to the wheelchair through the rectangular window
- **Attach** hook in the correct way, raise the wheelchair/cover
- When the wheels are at eye level, zip up the cover
- At this point (eye level) attach the locking strap (this may need adjustment)
- Continue to raise the hoist so locking strap tensions, and the wheelchair/ cover is completely loaded

### To Unload the Wheelchair with Cover

- **Press** down on the switch unit, when the bottom of the wheelchair/cover is at eye level, **unhook** the locking strap and unzip the cover
- **Move** the cover up a little bit so when the wheelchair/cover lands on the ground the wheelchair is not sitting on the cover
- Land the wheelchair/cover, **remove** hook through rectangular window
- **Retract** hook until the pivot arm is returned to its original position
- **Remove** the weather cover, **rotate** the wheelchair onto its front wheels
- **Unfold** wheelchair and transfer

## Frank's Engineering Pty Ltd

135 Bakers Road North Coburg Vic 3058 | Ph: 03 9354 0400 | Fax: 03 9354 0344

Email for Quotes: [sales@franksengineering.com.au](mailto:sales@franksengineering.com.au)

[www.franksengineering.com.au](http://www.franksengineering.com.au)